

SEER™ SET UP GUIDE



Visit us at: www.boviemed.com

SEER™ SALINE ENHANCED ELECTROSURGICAL RESECTION

MANUFACTURED BY BOVIE® MEDICAL



Note: Refer to complete Directions For Use (DFU) provided with the device prior to use.

Step 1:

Hang saline bag on the I.V. Pole at a height approximately 6-7 ft



Step 2:

Aseptically present the device to the sterile field.



Step 3:

Spike the saline bag using the I.V. set included



Step 4:

Connect the device plug to the monopolar output of the generator. Refer to the User Guide provided with the generator for further instructions.

Equipment needed to use the SEER Device:

- 1L sterile 0.9% NaCl Saline bag
- Standard I.V. Pole
- Electrosurgical Generator per the DFU
- Standard Monopolar Return Electrode Pad

Step 5:

Fully open the flow control on the handle to prime the tubing until saline is dripping from the tip and all air bubbles are removed.



Blue knob should be completely distal.

Step 6:

Ensure tip is entirely saturated with saline.



Step 7:

Adjust flowrate and Power for Physician-preferred settings. One typical setting is 100W on the PureCut mode and the flowrate is one audible/tactile click proximal from the starting position.

